

**NEW**

**Ingram's Patent Seamless Collar or Rim Teat,**

**THE BEST**

**Soothing Teat**  **in the World.**

**PATENTED**  
No. 22458  
Patented in France, No.  
220745. April 7th, 1892.

**IN ENGLAND**  
DEC. 23RD, 1891  
AMERICAN PATENT  
APPLIED FOR.

Made in the same sizes as the ordinary Teats, viz:—  
Small, Medium and Large.

**ADVANTAGES :**

- 1.—Will not collapse during suction.
- 2.—The Rim (AA) prevents the Teat swelling when in use.
- 3.—Entirely prevents air entering the mouth.
- 4.—The cylinder of the Teat being narrow, does not distend the lips of the child.
- 5.—The Rim or Collar (AA) prevents the Teat slipping out of the mouth.
- 6.—Its soothing properties are unrivalled.
- 7.—It is made of Absolutely Pure Rubber, Tasteless and Free from Smell.
- 8.—Acknowledged to be the Best Teat ever offered to the Public.

**NEW**

**Ingram's Patent**

**Seamless Collar**

**or Rim Teat.**

The best Soothing Teat in the world.

The above is also made up in the following styles of

**Soothing Toys....**







Fig. 30  
In two sizes,  
Small and Large.

Fig. 31

Fig. 32

**THE RESPIROREGENERATOR**

— OR —

**Perfect Inhaler.**

Patented in England, 16th August, 1892.

PATENT No. 14318

HEIGHT OF WATER. \_\_\_\_\_

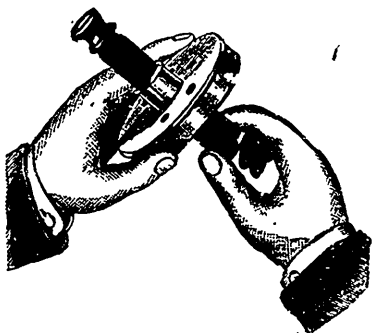
**Directions for using the Inhaler.**

Fig. 1



Fig. 2

1. Take the lid off the Inhaler and pass the mouth-piece through the hole from the inside, drawing it tight, as shown in Fig. 1.
  2. Remove the stopper of the glass bottle and pour the drug or medicine to be inhaled into the bottle, and place same in the Inhaler, as shown in Fig. 2.
  3. Fill the Inhaler with hot water up to the top of the perforated tube, replace the lid of the Inhaler, and apply the mouth-piece to the mouth and inspire or breathe in freely.
  4. If a strong vapour is required, pull the indiarubber tube closer down to the neck of the glass bottle containing the medicine.
  5. When again requiring to use the Inhaler, remove the stopper, and simply re-fill the Inhaler with hot water as before, or if more convenient, the water can be made hot in the Inhaler.
- N.B.—Procure the drug or medicine most suitable for your complaint from your own doctor.

**ADVANTAGES :**

1. When the patient has finished inhaling, the stopper of the bottle has only to be replaced, and no more of the drug is evaporated or wasted, which is a great advantage when expensive drugs are being used, as in ordinary inhalers the drug is mixed with the water, and consequently thrown away with it.
2. No mistakes can occur in the strength of the dose of drug or medicine, and it is immaterial how much drug is put into the glass bottle.
3. These advantages prove this Inhaler to be the most reliable and the most economical, and therefore the cheapest in the market.

**TO BE OBTAINED OF ALL CHEMISTS AND DRUGGISTS.**