



T-2 controls

Serum samples

REFERENCE CHART FOR THE ADDITION OF TEST SERUM AND STANDARDS

STEP No. 4: Addition of Test Serum and Standards to Wells:

- As illustrated in the diagram above and using a clean pipette, wash the microtiter plate (3x) by adding (6 drops per well) of Washing Buffer. Discard the used Washing Buffer in a Javex container.
- Add the pre-incubated materials (samples and standards prepared in Step No. 2 and No. 3) to the appropriate rows (3 drops per well). Row 3 to 6 (A-H) contain SB1 to SB4; Row 8-12 (A-H) should now contain X1 to X5.