


❖ INGRAM'S 1474 ❖

Imitation is the Highest Form of Flattery.

Owing to the many imitations of our Patent Enema, we are compelled to warn all who wish for a **GENUINE INGRAM'S ENEMA**, to refuse any that does not bear the No. 1474.

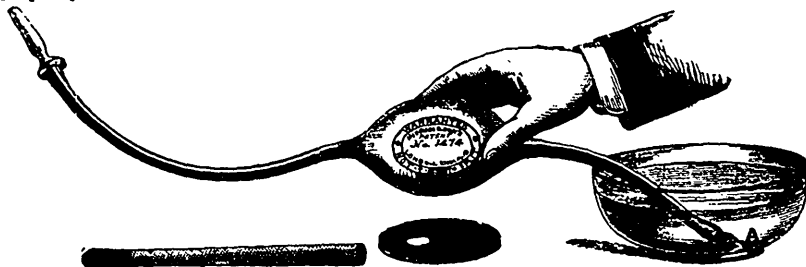

It will take years of practice for fresh hands to make this Enema—it being far more difficult to manufacture than the ordinary Barrel Enema.


J. G. INGRAM & SON have had 14 years' experience; therefore they do, with the utmost confidence, warrant every Enema of their manufacture bearing the number

❖ 1474 ❖

The New Back-Flow or Reverse-Current Ball Urethra SYRINGE.

Acknowledged to be the most efficient Urethra cleanser ever offered, as the action of the Back-flow washes, and drives out all foreign matter, instead of sending it inwards as with the old-fashioned Urethra Syringe.

EACH IN A NEAT BOX.

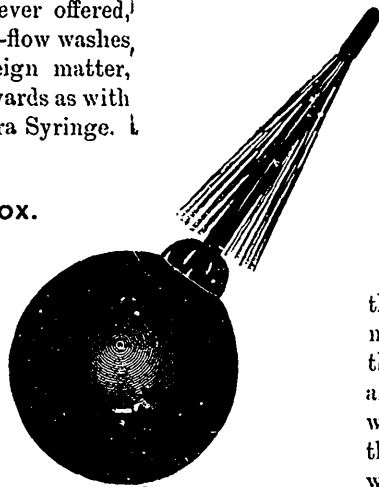

"Undoubtedly a Syringe of exceptional utility."

J. F. TAYLOR,
M.R.C.S., L.S.A., London



DIRECTIONS.

Tightly compress the Ball with the thumb and fingers, place the vulcanite pipe in the liquid, then release the Ball, which becomes quite full and prevents any air being injected with the liquid; insert the Pipe into the urethra and compress the Ball, when a perfect syringing and cleansing takes place.